bloomaligner™

# Congratulations!

By opening this box,  
you are restoring  
your smile and  
transforming  
your life.

Join the revolution in aligner development; and realize the power behind a **blooming smile.** Bloom™ adds to your happiness, and beauty and makes you more approachable. You have taken the first steps towards a contented look.

# Bloom™ creates life-changing smiles every day.

Bloom™ aligner technology; is the future of everlasting, healthy smiles. A modern-day technique made to move your life and your oral health to a higher level.

Bloom™ is a strikingly transparent, resilient, and advanced plastic aligner with a high level of comfort; created with great precision and pioneering, software; developed with orthodontics expertise and advanced engineering. Our superb aligner system is equipped with the best material using the best digital technology.

# Bloom™ gives our dear patients the self-confidence they deserve.

We strive, to give our dear patients the comfort they need during their correction phase. Our individually tailored, outstanding aligners, move teeth gently yet effectively.

A close-up photograph of a woman's face, focusing on her mouth and teeth. She is smiling broadly, showing her bright white teeth and vibrant red lipstick. Her hair is dark brown and styled in a shoulder-length bob. The background is a soft, out-of-focus light blue.

## Bloom™ clear aligner, for straighter teeth and a blooming smile.

Bloom™ aligner system is for all types of malocclusions. Our aligners are designed and developed by expert orthodontists for the aesthetic correction of your misaligned teeth. Our aligners are CE standards; created to improve oral health and provide good occlusion solving all types of Orthodontic Problems.

# How to apply your unnoticeable, biocompatible, and highly comfortable aligners.

Make sure you have the right upper or lower aligner first. The upper aligner is marked with the letter "U", and the lower aligner is marked with the letter "L".

## **Front to back**

Use this method if the anterior teeth are excessively anteriorly tilted (incisal edge forward) or excessively crowded.

- Insert either of the two aligners first.
- Attach the aligner to your front teeth
- Then push it onto your back teeth.

## **Back to front**

Use this method if all teeth are relatively vertical.

- Place the aligners on the back teeth first.
- Then push the tray onto your front teeth.

## Side to side

Use this method if the posterior teeth are excessively inclined towards the tongue.

- Attach to one side first, then attach to the front teeth.
- Then push to the other side.
- Carefully use your finger to push the aligner in.

## Anterior to backward

Use this method if your teeth are flared lingually or buccally.

- Attach the aligners to your front teeth.
- Then push the tray into the flared side. For example, if your teeth are lingually flared, push the tray lingually. If it flares buccal, tuck it buccally. (Apply the aligners to the most difficult areas first, and the remaining areas should be easy to apply.)

**\* Do not bite down on the aligners to set them in place. This could damage them.**

## How to change aligners ?

Remember to change your aligners within the time frame prescribed by your Dr. to abide to your treatment plan. Wear each set according to your Dr. recommendations; generally, between 20 to 22 hours per day.

## How to remove your aligners ?

Bloom™ is more difficult to remove than to wear. Don't use any force, just use your fingers to remove the aligner starting from the left or right back teeth. Gently remove the aligners from your molars. Do the same thing on the other side, then gently and delicately release the aligner from your teeth by sliding it out.

## How to care for your bloom™ aligner ?

Here are three ways to care for your mouthpiece.

- Remove your aligners when eating or drinking anything apart from water.
- Brush your aligners, using neutral toothpaste, a soft-bristled brush, and cold or lukewarm water, after meals or drinks. Floss your teeth.
- Soak your aligners in water when not used.

**Bloom™ serves tens of thousands of dentists around the world.**

Delivering hundreds of thousands of aligners to patients. Our clear aligners are CE approved and contain no BPA, BPS, latex, mercury, phthalate, or gluten. Enjoy the comfort of your bloom™ aligners.

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CEDARWINGS SAS