



LoocidBCP™ Drill Protocols

LoocidBCP™ Drill Bits used for Neodent implants GM Helix Conical

Neodent GM Ø	Loocid Starter Drill Bit	Neodent GM Helix
3.5mm	2.0mm	2.0mm
3.75mm	2.0mm	3.5mm
4.0mm	2.0mm	3.5mm
4.3mm	2.0mm	3.5mm
5.0mm	2.0mm	3.5mm

The LoocidBCP™ Drill Bits can be used in conjunction with the Neodent GM Helix Conical Implant system. The surgical sequence is:

- 1) LoocidBCP™ 2.0mm Starter Drill Bit
- 2) GM Helix Drill Bit:

The GM Helix Drill Bit following the use of the LoocidBCP™ Drill Bit is shown in the table above. Please finish the implant bed preparation used for the correct implant diameter and bone density by following the regular Neodent GM Helix protocol

